




Von-Werthern-Str.4a,
96487 Dörfles-Esbach, Deutschland

Tel: +49 89 215 400 980 / +49 95 619 769 199
Mail: de.info@donnerberg.net • Web: www.donnerberg.net

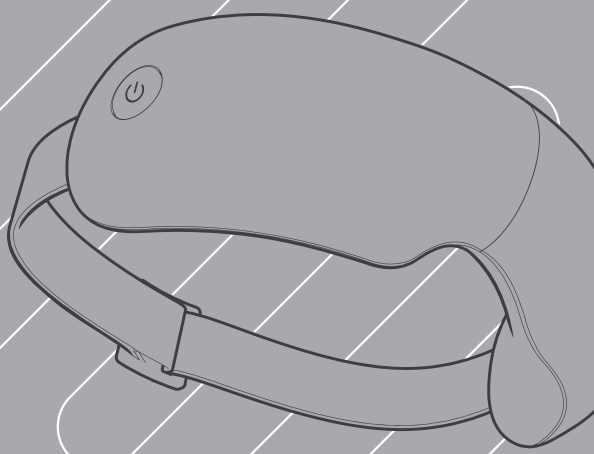

NEO
DREAM

BEDIENUNGSANLEITUNG

WWW.DONNERBERG.NET

Foreword

Dear user:
Welcome to use Smart Eye Mas sager. To ensure that you get the most from your massager,
read this manual thoroughly before use and keep it handy when using the product.

- 01 -

Contents

Important Safety Instructions	03
Packing	05
Preliminary Checks	06
Function Description	07
Operation Guide	08
Function modes	09
Transportation and Storage	10
Products Info	11
Troubleshooting	12
Warranty Card	13

- 02 -

Important Safety Instructions

⚠ Warning

- We recommend that those with any of the following conditions should consult your physician before using:
- Contact lens wearers, remove the eye contact lens before using this appliance.
 - Eye surgery patients at rehabilitation stage.
 - Those undergoing treatment and not feeling well.
 - Those with hypertension,ocular hypertension, arteriosclerosis or cerebral venous sinus thrombosis.
 - Those with high fever, malignant tumors,malignant abscesses, hypertension or osteoporosis.
 - Those with Infectious diseases.

- 03 -

Important Safety Instructions

⚠ CAUTION

- Turn off the power and unplug the product after use or before cleaning.
- Keep away from water, heat,open flame and corrosive environment.
- If it's accidentally dropped into the water while in charging, immediately remove the plug from the power socket and do not touch it with bare hands.
- Do not use this product in high temperature,high humidity or High electromagnetic fields areas.
- This product contains lithium-ion battery, do not attempt to repair or dismantle the appliance or put it in open flame to avoid danger.
- Do not overrun this product,it may cause overheat or discomfort to the massage parts.
- This product should not be used to treat symptoms of any disease or by invalids.
- Close supervision is necessary when this product is used by, on, or near children or invalids.
- Use this product only for its intended use as described in this manual.

- 04 -

Packing

Before using, please verify that all components are included(check with the packing list carefully),
contact the dealer from which the purchase was made if any item is damaged and/or missing.

Packing List::

01. Massager	*1
02. Type-C charging cable	*1
03. User manual	*1

- 05 -

Preliminary Checks

Please check the power, if battery level is low, appliance will shut down or won't start, please charge the battery and restart.Be sure to disconnect it from the charger when fully charged to avoid shortening of battery life.

Charging

- Plug the USB end of the power cord into the product charging port.
- The indicator will be lit red while charging and turns cyan when fully charged.
- Device does not work while charging, self-protecting process will start automatically when charging starts.
- This massager uses built-in rechargeable battery, complete charge will take about 3 hours.
- Voice alerts will be heard when batteries are running low, please stop using,charge the battery and restart.


Versatile Charging Methods

Computer USB port / portable power supply / 5V=1A power adapter

Note: The default massage time is 15 minutes, you can restart massage by pressing the switch.

- 06 -

Function Description

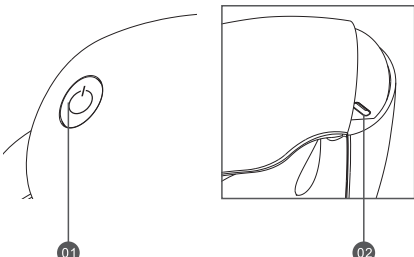

Functional diagram:
01. Power/Mode/Music

- | | | | |
|----|------------|----|-------------------|
| 03 | Music icon | 05 | Charging icon |
| 04 | Music icon | 06 | Hot compress icon |
- 02.Type-C charging port
03. Music indicator light
04. Air pressure indicator light
05. Charging indicator light
06. Hot compress indicator light

- 07 -

- Hold down the button for 2 seconds to turn it on/off. When playing music, double-click the switch to stop playing music, and double-click the switch again Resume the music playing, and press to switch working modes. There are four working modes: comprehensive mode, eye love mode, eye caring mode, sleep mode.
- When the battery power of the product is low, the product will give a voice alarm of "low battery".
- The product will automatically shut down when it is charged, and the product cannot be powered on when it is charged.
- Manual pairing and connection is required for the first time with Bluetooth devices that have never been connected to the product. The product's Bluetoothname is Eye Massager. Devices that have previously been connected before will automatically reconnect to the Bluetooth when the product is turned on.

- 08 -

Function modes



- 09 -

Transportation and Storage

Environment temperature: -10℃-65℃

Relative humidity: ≤80%RH, well-ventilated room containing no corrosive gases.

This product is equipped with simple anti-impact facility for transportation, please keep the box away from rain or snow, avoid colliding and do not put upside down. When left in idle conditions, appliance should be checked and recharged at an interval of not more than six months.

Keep the product away from direct sunlight or corrosive gases.

- 10 -

Products Info

Product Name: Smart Eye massager
Product model :C10
Rated input :5V ---
Rated power :5W
Battery capacity :1200mAh
Charging interface :Type-C
Product size :L209mm*W108mm*H72mm

- 11 -

Troubleshooting

If you are having trouble using the product, please try the following: If problem remains, please turn off the power, do not attempt to repair or dismantle the appliance and contact the authorized dealer for maintenance.

Symptoms	Possible Causes	Possible Solutions
Appliance does not start	Low battery	Charge the battery and restart
Appliance won't charge	The USB charging wire is not well connected	Plug the USB charging wire properly
	Wrong charging wire	Use the original charging wire
Overtime charging	Power adapter with mismatched current	Change the power adapter with ≥ 1A current
No music playing	Music pause state	Double click on the switch to resume playing music

- 12 -

Warranty Card

Warranty Card	
Name:	
Phone:	
Address:	
Email:	
Product Name:	
Date of Purchase	

Warranty Date	Warranty Cotent	Comments

- 13 -